
26.01.2025

Brukermanual / User Manual

FORUS LEDSAGERSTYRING /

FORUS ATTENDANT CONTROL

- HP20071066

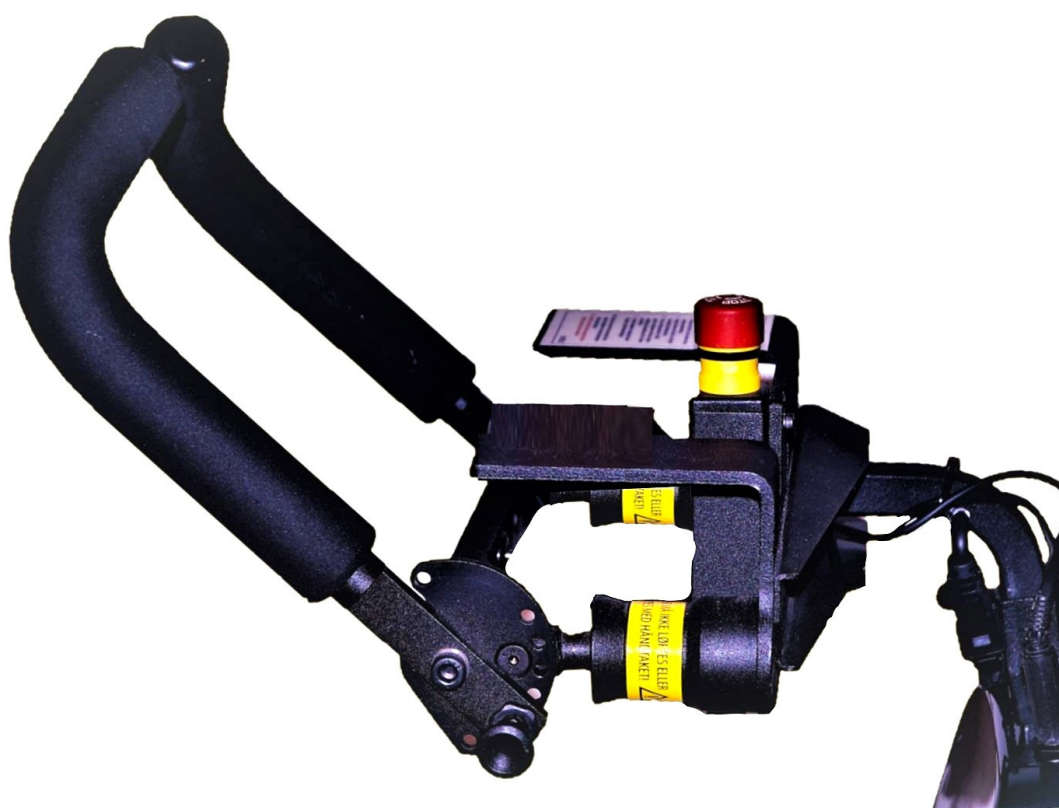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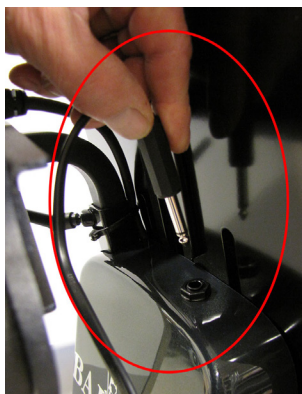
Figur 8. Hendel på ledsagerstyring venstre side

Figur 8. Fjern ledsagerstyring

Using the attendant control with emergency stop

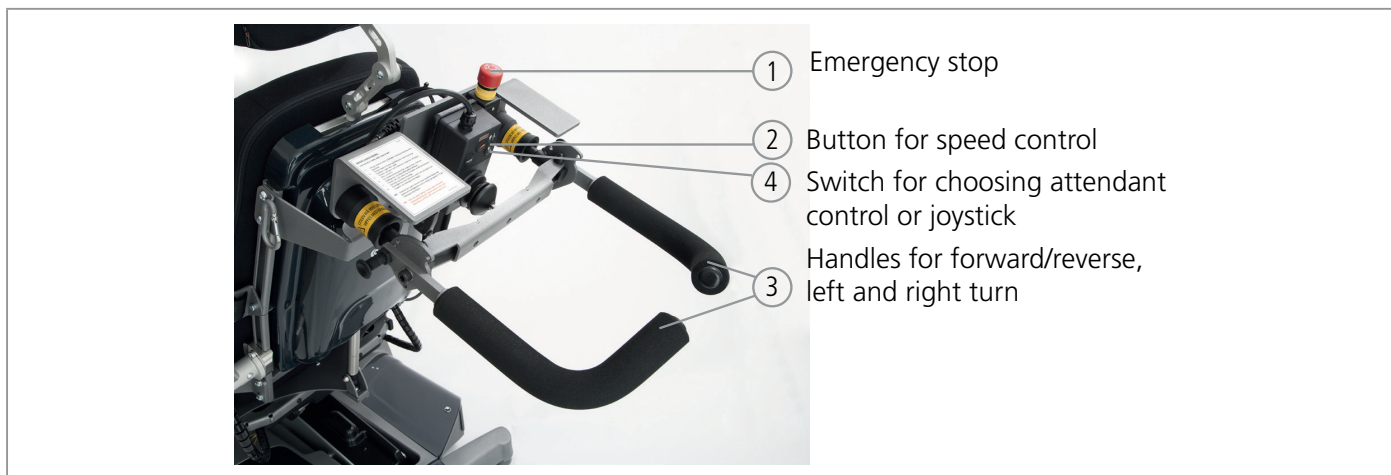
Figure 1. Attendant control with emergency stop, E800

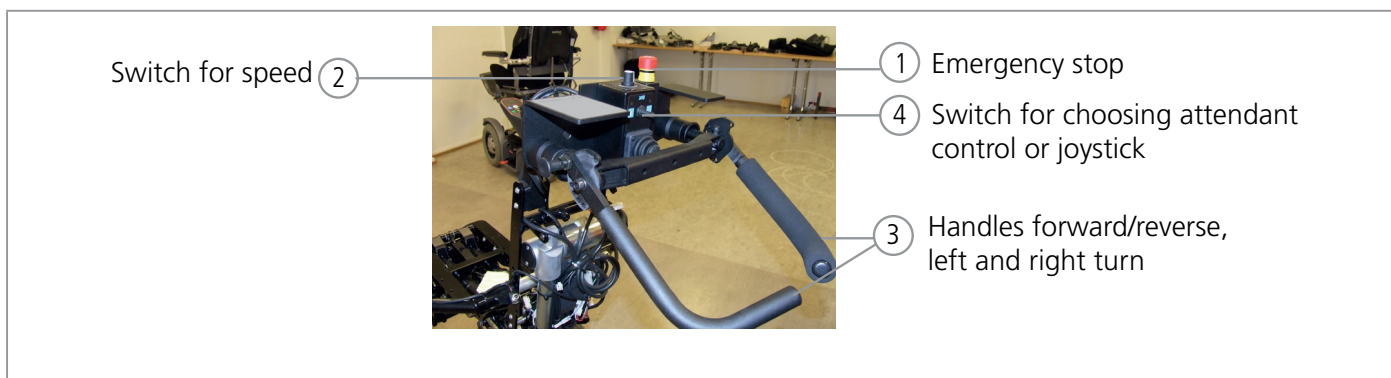
Figure 2. Attendant control with emergency stop, Balder

- Activate the ON / OFF button, choose between attendant control or joystick. (Pos 4)
- The chair runs forward and reverse by moving the handles up or down. Turn by moving the handles to the right or left.
- The chair must be stationary to switch between attendant control and user management.
- Speed is adjusted by pressing / turning the button in front of the attendant control.

Note: The wheelchair may be delivered without joystick in front. Speed is then adjusted by selecting driving program on the assistant control.

Important

Never lift, attach or load the attendant control unit or handle bars other than described.

Emergency button is located on top of the attendant control. The button is marked with a red color. Press the button in the event of a situation in which the chair must be stopped immediately. Release the emergency stop by turning clockwise

WARNING

Adjust the push handles downwards, to avoid the risk of being crushed or trapped by driving indoors or in tight passages.

Figure 3. Correct use of attendant control in order to avoid the risk of getting trapped

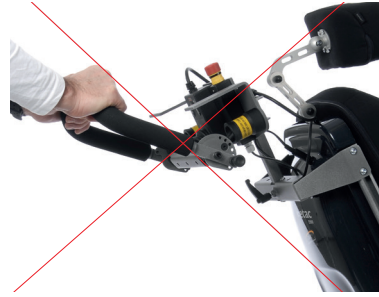
Figure 4. Incorrect use of attendant control indoors and in narrow passages

Dismount / mount removable attendant control

- Deactivate the chair by using the ON/OFF button on the joystick
- Remove the plug from the attendant control the chair's seat back (right hand side)
- Loosen the lever by turning it counterclockwise (left hand side)
- Lift/remove the attendant control
- To mount the attendant control, operation is repeated in reverse order

Figure 5. Removable attendant control w/emergency switch, E800/Balder

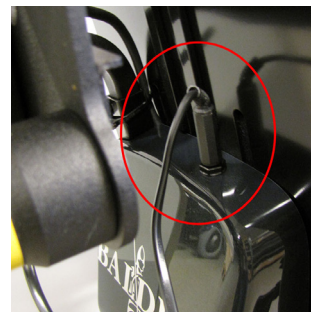
Figure 6. Connector on the right hand side

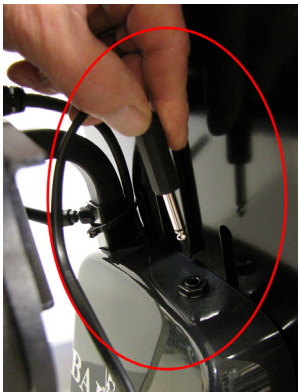
Figure 7. Remove connector

Figure 8. Lever on the left hand side

Figure 8. Remove attendant control

Label serial number

Baldertech AS
Email : info@baldertech.com
Telephone : +47 32 11 11 00
www.baldertech.com

baldertech