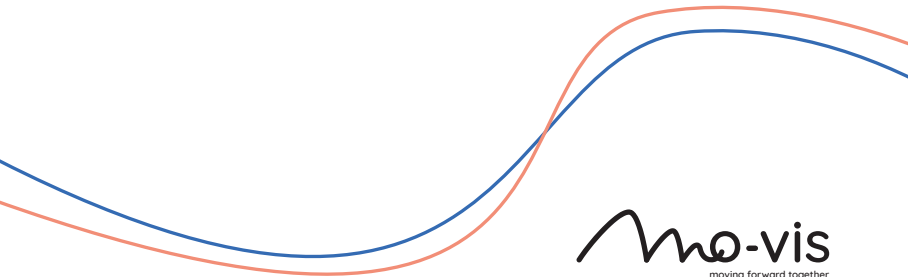
User manual

Edition D-P015-61-70-M9-0700, August 2023

Scoot Control

P015-61 Scoot Control R-net - P015-71 Scoot Control LiNX

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EN: You can find this manual in English on our website.

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You can find the latest version of all our manuals on the website.

About this manual

User manual

Thank you for choosing a mo-vis product!

This manual contains **useful and important information** about your device.
Please read it carefully before use and store safely for future reference.

Our team (or your authorized dealer) will be happy to answer your questions.

mo-vis bv

Biebuyckstraat 15D . 9850 Deinze .
Belgium

<http://www.mo-vis.com> .
contact@mo-vis.com . +32 9 335 28

60

CH	REP
----	-----

CH-REP: SKS Rehab AG, Im Wyden,
8762 Schwanden GL, Switzerland.

**UK
CA**

UK Responsible Person: QServe
Group UK, Ltd., 49 Greek Street,
London, W1D 4EG, UK.

User manual

Warning labels

Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.

NOTE: This symbol indicates general notes and information.

CAUTION: This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.

WARNING: This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

Other labels:

Catalogue number: indicates the manufacturer's catalogue number so that the medical device can be

	identified.
	Batch code: indicates the manufacturer's batch code so that the batch or lot can be identified.
	Medical device: indicates that the item is a medical device.
	Date of manufacture: indicates the date when the medical device was manufactured.
	Serial number: indicates the manufacturer's serial number so that a specific medical device can be identified.
	Consult instructions for use or consult electronic instructions for use: indicates the need for the user to consult the instructions for use.
	Keep dry: indicates a medical device that needs to be protected from moisture.
	Do not use if package is damaged and consult instructions for use:

indicates that a medical device should not be used if the package has been damaged or opened and that the user should consult the instructions for use for additional information.

CE label: indicates that the manufacturer or importer affirms the good's conformity with European health, safety, and environmental protection standards.

WEEE: indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.

Manufacturer: indicates the medical device manufacturer.

Support, scrapping & recycling

Technical support

TROUBLE: In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: contact@mo-vis.com or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

Spare parts and accessories

Contact mo-vis or your dealer for more information about spare parts and accessories.

Scrapping & recycling

CAUTION: For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

Limited liability

*mo-vis accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings and instructions in this manual.*

CAUTION: This product should only be installed by a qualified service engineer.

NOTE: All mo-vis user manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.

NOTE: In case any serious incident occurs in relation to this device, this should be reported immediately to mo-vis and the competent authority of the Member State in which the user is established.

Intended use

The Scoot Control is a steering device which can be connected directly to the wheelchair electronics (**R-net** and **LiNX**), to control the power chair and its functions. It is intended to support an attendant in controlling or maneuvering a power chair, both inside and outside.

Safe driving

WARNING: The on/off switch must be available to the user at all times. This allows to instantly stop the wheelchair in case of problems or an emergency.

WARNING: In case the wheelchair responds in an unexpected way, the user must immediately release the Scoot Control or use the power on/off switch.

WARNING: Only adult attendant should operate the Scoot Control.

WARNING: When the wheelchair is in tilt/recline, please be aware that the Scoot Control could collide with objects or people (especially if the Scoot Control is not folded in).

Features

*The Scoot Control is part of our range of input devices. It is to be mounted on the back of the power chair and is a steering device with 2 handles (similar to a compact bicycle steer). **The Scoot Control is compatible with either***

the R-net electronics of Curtiss-Wright or the LiNX electronics of Dynamic Controls.

- Fully proportional handlebar with 2 rubber handles and 2 configurable thumb throttles at either side
- Built in a compact housing, with a 3.5 mm stereo jack to connect an external actuator keypad
 - P015-61 Scoot Control R-net: actuator keypad should be compatible with the Curtiss-Wright CJS2M2 resistor values
 - P015-71 Scoot Control LiNX: actuator keypad should be compatible with LiNX input control systems
- Full access to wheelchair functions such as Lights, Horn and Speed settings
- Battery (only R-net version) and speed display via LEDs
- On-board configuration possible
- Compatible with multiple types of power chairs equipped with R-net or LiNX electronics

Operation

The Scoot Control consists of the following parts:

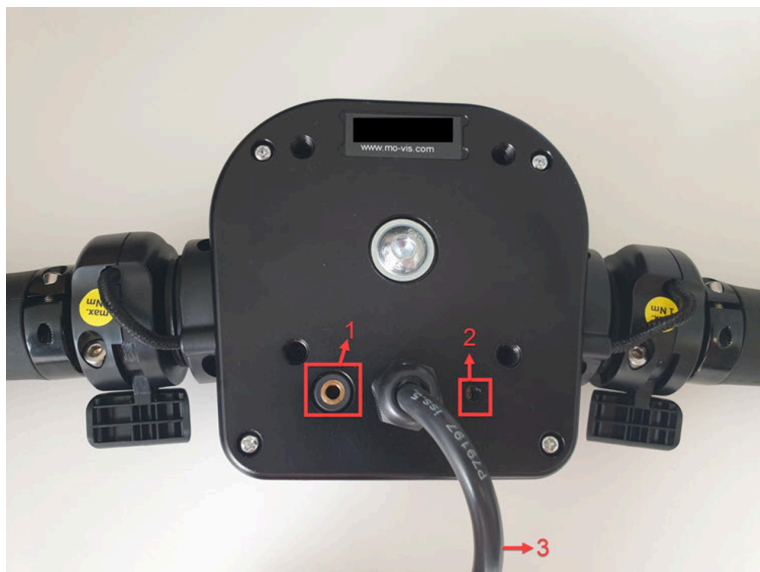
- 1 Control panel: control the Scoot Control

NOTE: The control panel is slightly different for the R-net and the LiNX version.

- 2 Handlebar: rubber handle at either side to steer the wheelchair
- 3 Left and right throttle: to move the power chair forward and/or backward

- 1 3.5 mm jack to connect an external keypad
- 2 Built-in sounder: to be used as horn
- 3 R-net or LiNX bus cable

Horizontal movements of the **handle bar** will result in a steering action of the wheelchair (turning).

In combination with the **thumb throttle** it is possible to move the wheelchair in any direction.

- Standard configuration: right throttle = forward, left throttle = backward
- If both throttles are operated simultaneously and that would result in conflicting direction, the chair will stop driving

WARNING: Do not hang things (e.g. backpacks) on the Scoot Control or the throttle. This might damage the throttle and the device. Furthermore, removing items suddenly from the throttle might lead to unintended movement of the power chair if it is not in stand-by mode.

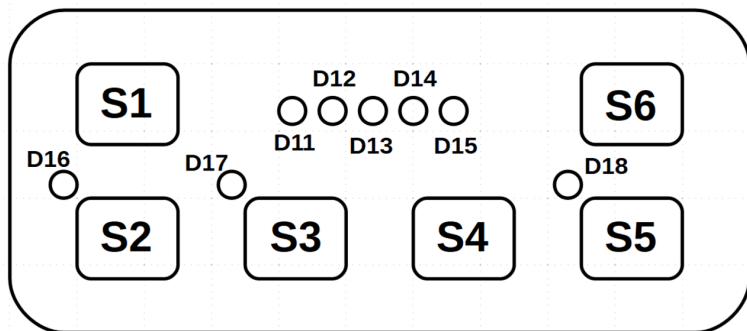
CAUTION: Pushing harder on the throttle will not increase the speed of the power chair. To increase the speed, press S6 or change the settings of the power chair.

CAUTION: When you hold the left or right throttle down for more than 1 second during the stand-by mode, a beep will sound. This is to alert the attendant that the Scoot Control is in stand-by mode and that the wheelchair will not react to the throttle movement. The user has to release the throttle in order to control the power wheelchair again.

Control Panel R-net version

The **buttons** have the following functions:

BUTTON	FUNCTION	ACTION
S1	On / Off	Switch the power chair on/off
S2	Left indicator	Switch the left indicator on/off
S3	Right indicator	Switch right indicator on/off
Hold S2 for > 2 seconds	Hazard lights	Switch hazard lights on/off
Hold S3 for > 2	Lights	Switch lights on/off

BUTTON	FUNCTION	ACTION
seconds		
S4	Horn	Horn sounds as long as pressed
S5	Direction	Switch direction of the throttles
S6	Speed setting	Change the speed. The first click on S6 will show you the current speed setting. The next clicks will change the speed.
Hold S6 for > 2 seconds	Grab focus	Press long to grab control of the power chair. See <i>Scoot Control Installation Manual</i> for more information.


LED indications on the control panel provide you with the following information:

LED	FUNCTION	INFORMATION
D11-D15	Battery gauge	Battery indicator: is default on when Scoot Control is in focus
	Speed setting	Speed indicator when setting the speed. 5 speeds = 5 green LED in a row (e.g.

LED	FUNCTION	INFORMATION
		speed level 3 = 3 LEDs on) Becomes active when pressing S6 and remains active for 5 seconds after last speed change.
	Out-of-neutral	When the device is in out-of-neutral, D13 will be lit continuously. Other LEDs are off.
	Error	D11 and D15 red, D13 flashes red, other LEDs are off: number of flashes indicates the fault (see <i>Scoot Control Installation manual</i>).  NOTE: Only mo-vis faults are shown, R-net system faults are shown on the remote.
	Status	• In focus: battery indicator • Not in focus: D13 green heartbeat flash • Configuration: D13 green fast flash
D16	Left indicator	Green flash when left indicator is on
D17	Right indicator	Green flash when right indicator is on
D16 + D17	Hazard lights / lights indicator	Both flash red when hazard lights are on.

LED	FUNCTION	INFORMATION
		Both are orange when the lights are on and the direction indicator is off.
D18	Forward / reverse	In standard configuration: Forward = green Backward = orange
	No focus	Off
	Stand-by	Flash green / orange (depending on direction of throttle)

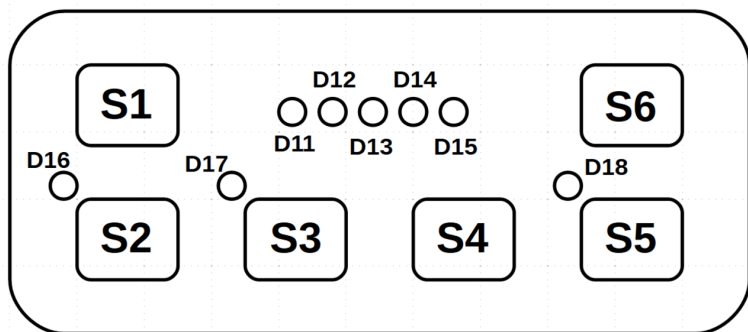Control Panel LiNX version

The **buttons** have the following functions:

BUTTON	FUNCTION	ACTION
S1	On / Off	If the wheelchair is off when S1 is pressed, the system will turn on with the attendant in charge. If the wheelchair is on when S1 is pressed, the system will hand over control to the attendant. When you press S1 again, you turn the wheelchair off.
S2	Hazard lights	Switch hazard lights on/off
S3	Lights	Switch lights on/off
Hold S2 for > 2 seconds	Left indicator	Switch the left indicator on/off
Hold S3 for > 2 seconds	Right indicator	Switch the right indicator on/off
S4	Horn	Horn sounds as long as pressed
S5	Direction	Switch direction of the throttles
S6	Speed setting	Change the speed.

LED indications on the control panel provide you with the following information:

LED	FUNCTION	INFORMATION
D11-D15	Speed setting	Always show the speed, when Scoot Control is in focus. 5 speeds = 5 green LED in a row (e.g. speed level 3 = 3 LEDs on)
	Out-of-neutral	When the device is in out-of-neutral, D13 will be lit continuously. Other LEDs are off.
	Error	D11 and D15 red, D13 flashes red, other LEDs are off: number of flashes indicates the fault (see <i>Scoot Control Installation manual</i>).  NOTE: Only mo-vis faults are shown, LiNX system faults are shown on the remote.
	Status	• In focus: speed setting • Not in focus: D13 green heartbeat flash • Configuration: D13 green fast flash
D16	Left indicator	Green flash when left indicator is on


LED	FUNCTION	INFORMATION
D17	Right indicator/ Lights	Green flash when right indicator is on. Solid green when lights are on.
D16 + D17	Hazard lights	Both flash green when hazard lights are on.
D18	Forward / reverse	In standard configuration: Forward = green Backward = orange
	No focus	Off
	Stand-by	Flash green / orange (depending on direction of throttle)

Parts and accessories

The Scoot Control is always accompanied by the *D-P015-61-70-vv Scoot Control User manual* and the *D-P015-61-71-vv Scoot Control Installation manual*.

The following parts and accessories are **optionally** available:

PART NUMBER	FUNCTION	INFORMATION
P016-98 Actuator	You can connect an	See <i>Scoot Control Installa-</i>

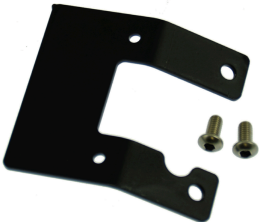PART NUMBER	FUNCTION	INFORMATION
Keypad Button R-net CJS2	external Curtiss-Wright CJS2 compatible Actuator Keypad to the Scoot Control R-net (P015-61) to control the actuators of the power chair.	<i>tion Manual</i>
P016-99 Actuator Keypad Paddle R-net CJS2		
P016-88 Actuator Keypad PRO Button R-net CJS2		
P016-89 Actuator Keypad PRO Paddle R-net CJS2		
M015-91 Scoot Control keypad bracket set	To mount an Actuator Keypad to the Scoot Control	

PART NUMBER	FUNCTION	INFORMATION
		See Dimensions on page 32
M015-70 Scoot Control bracket set	Dedicated mounting part for the Scoot Control	Available together with Scoot Control as P015-65 Scoot Control R-net bundle or P015-75 Scoot Control LiNX bundle. See Dimensions on page 32 
M015-71 Scoot Control bracket set short	Dedicated mounting part for the Scoot Control	Available together with Scoot Control as P015-66 Scoot Control R-net short bundle or P015-76 Scoot Control LiNX short bundle.

PART NUMBER	FUNCTION	INFORMATION
		See Dimensions on page 32 
P015-51 Scoot Control Throttle R Assembly	Right throttle	
P015-52 Scoot Control Throttle L Assembly	Left throttle	
P015-53 Scoot Control Throttle Set	Right and left throttle	

Contact mo-vis or your dealer for more information about spare parts and accessories.

Qualifications

CAUTION: Only a qualified service engineer may install the device.

CAUTION: An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

First time use

CAUTION: A qualified service engineer should assist during first time use of the device and make sure that all functions are clear to the user, including what to do in case of problems or doubt.

Cleaning

Clean all parts of the Scoot Control on a regular basis (monthly) or whenever needed.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.

CAUTION: Do not immerse in water or use excessive amounts of liquid.

CAUTION: Do not apply additional lubricants to the moving parts.

CAUTION: Pay special attention to the cleanliness of the throttles. Dirt and liquids could lead to reduced functionality of the Scoot Control.

Warnings and limitations

NOTE: This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

WARNING: Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

CAUTION: If an output inhibit is programmed, then it should be checked monthly if this is still operational. Please see the *Installation Manual* for more information.

Dimensions

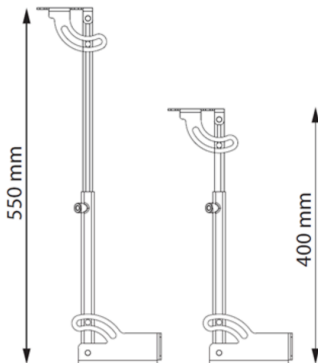
Scoot Control

450 x 100 x 55 mm - 17.72 x 3.94 x 2.17 in (WxDxH)


Mounting Brackets

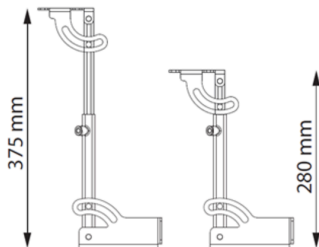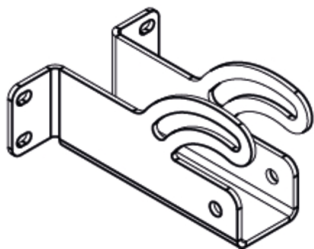
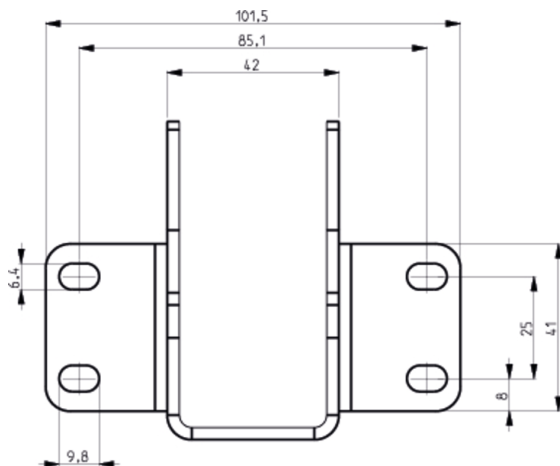
- M015-70 Scoot Control bracket set
- M015-71 Scoot Control bracket set short

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