


23.08.2025

## BALDER

- Finesse
- Junior
- Liberty

Maintenance, cleaning and recycling

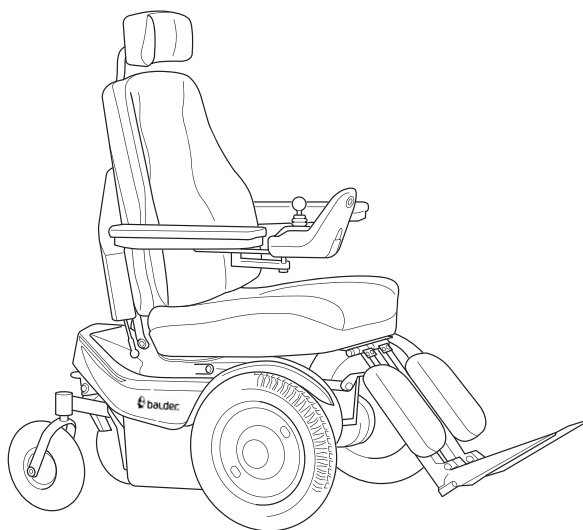

This manual contains information about maintenance, cleaning and recycling of Balder wheelchairs. Relevant information is otherwise distributed in the user manual which comes with the product at delivery, as well as in the technical handbook.

## **Maintenance**

There are no requirements to service or changing of parts attached to maintenance of the wheelchair. However, in order to keep the quality of the product and its properties, appropriate use of the product is provided as well as regular inspections, maintenance and cleaning when needed.

Besides checking the functions of the chair, safety equipment, charging of batteries etc., we recommend service intervals of every 2500 hours of use / once a year / when needed.

## **Cleaning**

The wheelchair is cleaned when needed. There are no requirements when it comes to tools needed to clean the product. Chemicals or strong detergents should not be used, nor should brushes or other equipment that may damage the product. Use mild soapy water without abrasive and equipment that do not damage the product.

Seat:

- Can be soak cleaned using upholstery cleanser
- Water gently and vacuum
- Vacuum and dry
- See washing instructions on the seat cover for more information about washing and drying

The rest of the chair:

- Remove larger objects by hand
- Blow off dust etc. by using a blowgun gently. Make sure that it is not blown into electrical components/contacts
- Wipe off the parts of the chair using a cloth moistened in gentle soapy water
- The chair should be placed on a level surface to air dry

## **Disinfection**

- Approved disinfectant for surfaces can be used

NB! After cleaning, all movable items that are oiled originally should be oiled again with lubricants of low viscosity such as 5-56, WD40, CRC or others.

NB! The chair must never be washed down using a high-pressure washer or running water.

## **Recycling – preparing for re-use**

When the wheelchair is up for recycling, it is important that it undergoes a technical control and is properly cleaned before it is delivered to a new user. There are no requirements to changing of parts, but if a technical control proves this necessary, this must of course be done. Of hygienic reasons, we recommend that cushions and covers are changed at recycling. For information about prices, see current pricelist for the product.

See next page for check list for periodic inspections/preparing for re-use.

## Maintenance routines

Check list for periodic inspections/preparing for re-use. The document can be found in the technical handbook.

Chassisnr.: ..... Date of execution: .....

Aid no.: ..... Individual nr.: .....

Description	Area	OK	Replaced	Adjusted	Repaired	Sign.
Visual check of the chair	Body moulding					
	Lights					
	Lights					
	Tyres					
Control of the functions	Electrical functions					
	Driving characteristics					
	Speed restrictions					
Mechanical Control	Rear/front carriage					
	Circlips					
	Set screws					
	Drive motors					
	Magnetic brake					
	Micro Switch					
	Actuator brackets/ fixings					
Grease points	Back actuator fixing bolts					
	Lift actuator screw					
	Lift actuator lower bolt					
	Length comp actuator screw					
	Footrest arms					
Electrical Control	Electronic fault log					
	Batteries					
	Cables					
	Charger					
	Main cables					
	Connections power/actuator unit					
	Connection actuators					
Miscellaneous	Wash and clean					
	Adjustments for the user					
	Users manual					

Executed by: ..... Signature: .....

**Baldertech AS**

Email : [info@baldertech.com](mailto:info@baldertech.com)  
Telephone : +47 32 11 11 00  
[www.baldertech.com](http://www.baldertech.com)

**baldertech**